


Examination Table Leg

Fitting Instructions

Locate legs and brackets on the bottom of your table top – as shown. Table top should be 18mm thick (or more).

Brackets should be either side of the tube dimples and located a minimum of 10mm from the edge of the top.


The captive end of the brackets (3 screw holes) should be closest to the ends.

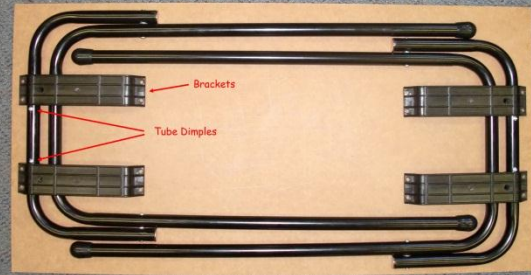

Insert and screw down the long screws into the holes shown.



Insert and screw down the short screws into the holes shown.

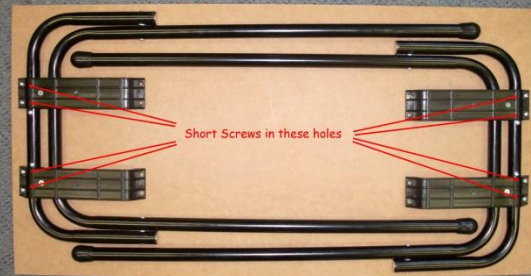

Hold Bracket down and screw in remainder of short screws.



Ensure the 4 long screws and 16 short screws are all in place



Your legs are now ready – open them up and stand up your completed table.

